Medical Use Nonwoven Disposable 3ply Medical Surgical Face Masks With CE certification




3 Layers

Skin-friendly composite fiber

High density filter layer

Non-woven fabric

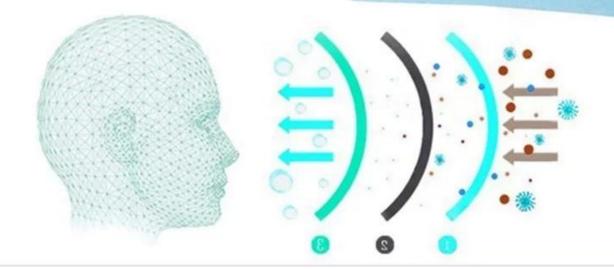

How to wear a mask correctly?



Note: The part with the metal strip should be above the mask

Distinguish the front and back of the mask and up and down, then wash your hands





After wearing the mask, squeeze the metal strips on both sides of the bridge of the nose.



Then stretch the mask down to cover the nose and mouth so that the mask does not leave wrinkles.



Magic nose bar


Adjust the face fit to ensure the protection effect.




Elastic elastic belt Loose design, Comfortable to wear



Three fold, large space comfortable

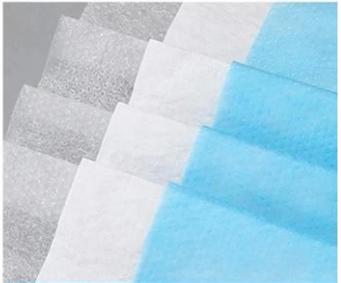

Three layer filter design
Use non-woven fabric, melt blown cloth and other filter materials.

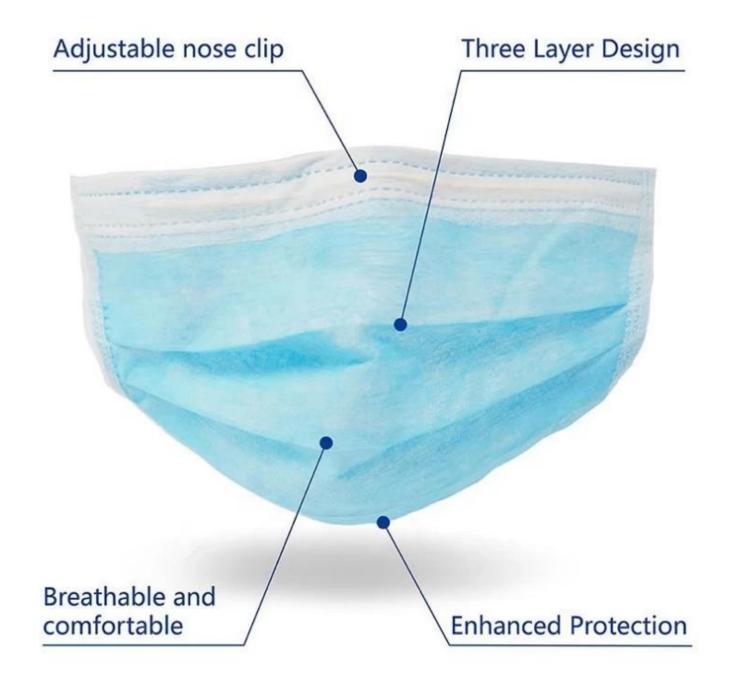

Detailed pictures

company information

About us Lotus Umbrella:

Shaoxing Lotus Umberlla Co., Ltd is a professional manufacturer of Amberla in Zhejiang Province, China. We have worked in this field for about 19 years and we offer a variety of custom umbrellas, including time, various sizes, fabric styles and colors of umbrellas, children, umbrellas, bottles You can provide full quality umbrellas which include straight umbrellas, folding umbrellas, golf umbrellas,

umbrellas, beach umbrellas, promotional umbrellas. We also have a professional design team and provide personalized and free design services.



OEM PROCESS



(02 E **03**

04

05

artwork from

make the quotation

make the sample for you confirmed place the order

mass production inspection packing shipment

PRODUCTION PROCEDURES



1. EDGE STITCH




2. CUT FABRIC



3. SEWING



4. TIP ASSEMBLY

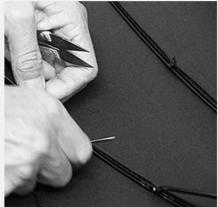

5. SEW UP



6. QUALITY CHECK

BUSINESS PROCESS









EXHIBITION

Packaging and shipping



our service



frequently asked Questions

1. Are you a manufacturer or a trading company?

Of course, we are a professional manufacturer with more than 19 years experience.

2. Can I provide free samples? Sampling time

The first time he collaborated, he would collect a small sample fee for the cost of setting up the project. Your order will be returned upon confirmation. You can send the sample within 5 days.

What are the payment terms?

T / T: 30% deposit, 70% balance against the copy of B / L

How long does it take to produce?

ETD: within 25-35 days after order confirmation

5. How do we manage the quality in the factory?

- I. We have strict quality control from the approval of each material until the end of the umbrella. In general, see AQL $2.5 \ // \ 4.0$.
- B. Samples made for approval from vendors before production We carry out strict online quality control in every production process.
- lii When the item is packed in a box, our quality control will use AQL to inspect the product at random and send the report.